



Prestan Professional Manikins Replacing the Prestan Infant Rate Monitor

To replace the Rate Monitor on the Adult Manikin, you will need a small Flat Head Screwdriver.

Remove the Monitor Cover (located in the diaper area of the infant manikin) by gently inserting the edge of the flathead screwdriver into one of the tabs of the monitor cover. The cover should pop out easily.

Remove the old Rate Monitor by lifting it straight out of the cavity.

Insert the new Rate Monitor into the cavity and press the cover back onto the diaper area. The lights may begin to blink, but can be shut off by pushing the On/Off Button.

